

Prevent pregnancy with ease.
Lady-Comp basic – the intelligent fertility computer.



LADY-COMP® basic

Lady-Comp basic – simple and without side effects.

Safe and reliable from day one with 99.3% accuracy.

The program.

A woman can only become pregnant on six days during each menstrual cycle: five days before ovulation and on the day of ovulation. Based on your morning temperature, taken orally, Lady-Comp basic determines your fertile days and shows when you are ovulating by recording, analyzing and storing your menstrual cycle data. It determines your daily status of fertility and will alert you on the days when intercourse may lead to pregnancy.

It's so easy!

- 1 Take your temperature under your tongue every morning.**
- 2 Enter menstruation on the days you have them.**
- 3 Lady-Comp basic evaluates your data and calculates your fertility status for the next 24 hours:**

Reliability is most important.

Lady-Comp basic's daily fertility display has a Pearl-Index of 0.7, meaning it is 99.3% accurate. The fertility computer is the uncontested leader in terms of accuracy, ease of use and safety.

The Pearl-Index is a value in the medical field that is used to determine the level of accuracy of a given birth control method. For example, if 100 women use contraception for one year and one of them becomes pregnant, then this corresponds to a Pearl-Index of 1.0.


Upgrades.

If you wish to upgrade the software or add more functions, you can do this at any time via an SD card.



- = Infertile
- = Learning phase
- = Fertile
- ☀ = Ovulation

Medical basics.

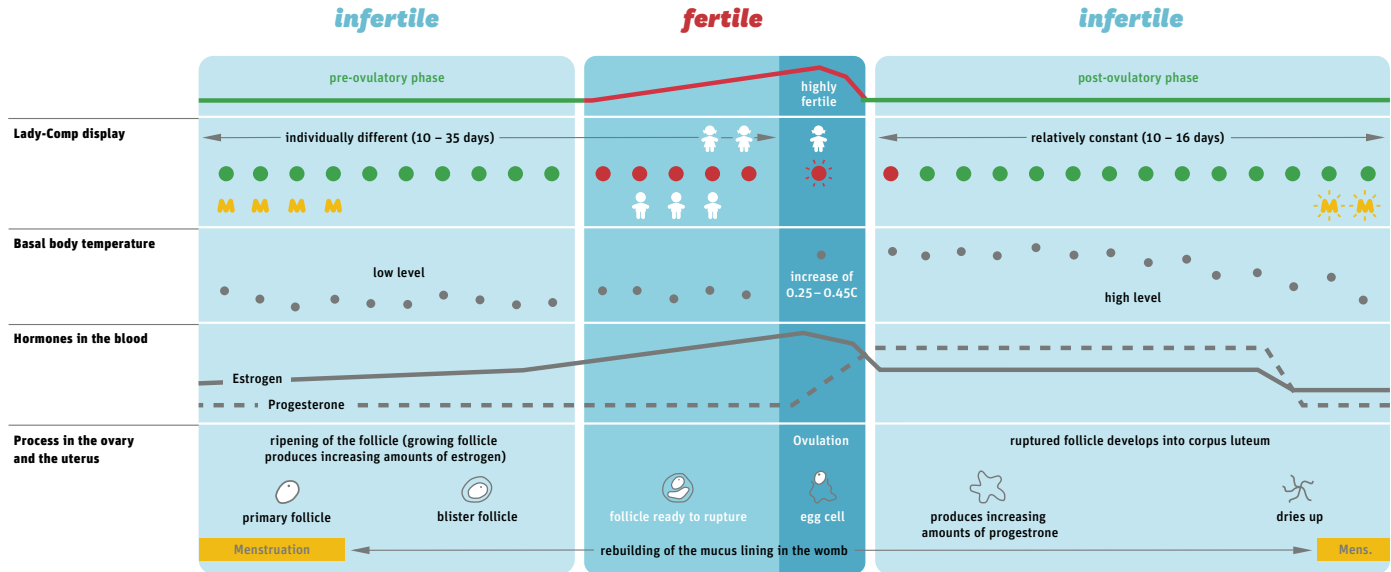

Biological facts:

- Every woman ovulates only once each cycle.
- If on the rare occasion two or more ovulations occur, they will occur within 24 hours.
- After ovulation, the egg can be fertilized for a maximum of 18 hours.
- Sperm can remain active and fertile in the woman's body for a maximum of 5 days.

That means:

Just on six days during each cycle pregnancy can occur, five days before ovulation and on the day of ovulation.

Lady-Comp basic knows:

Lady-Comp basic uses these biological facts and your individual cycle to reliably indicate when you are fertile or not.

More information.

For current prices and detailed information, please visit: www.lady-comp.de



**DEVELOPMENT, MANUFACTURING,
WORLDWIDE DISTRIBUTION**

 VE Valley Electronics GmbH

Breite 2

82418 Murnau/Germany

Tel.: +49 (0) 8841 48928-0

Fax: +49 (0) 8841 48928-24

info@lady-comp.de

www.lady-comp.de